

Reinigungstipps vom Fachmann für HOSTAPHAN[®] FACE Shields

"Die Folie kann mit warmen Wasser und milder Seife gespült werden. Dabei und beim anschließenden Trocknen darf aber nicht über die Folie gerieben werden, da dies sehr leicht Kratzer hinterlassen kann. Ich empfehle, das Wasser ablaufen zu lassen und verbleibende Tropfen mit einem saugfähigen Material, wie Löschpapier, sauberen Baumwolltuch oder Schwammtuch abzutupfen. Man kann auch mit Aceton nachspülen, aber das ist aus Umweltgründen doch eher zu vermeiden.

Glasreiniger eignen sich, aber auch Desinfektionsmittel mit Lösungsmittelanteil können verwendet werden."

Dr. Klaus Petersen



Mitsubishi Polyester Film GmbH
Manager
Marketing Technical Service Industrial Markets

Cleaning Tips from the Expert for HOSTAPHAN[®] FACE Shields

"The film can be rinsed with warm water and mild soap. However, do not rub the film when doing so and when drying it afterwards, as this can easily leave scratches. I recommend to let the water run off and to dab off remaining drops with an absorbent material like blotting paper, clean cotton cloth or sponge cloth. It is also possible to rinse with acetone, but this is rather to be avoided for environmental reasons.

Glass cleaners are suitable, but disinfectants with solvent content can also be used."

Dr. Klaus Petersen



Mitsubishi Polyester Film GmbH
Manager
Marketing Technical Service Industrial Markets